

# **Hemaray 83**

5-part Hematology Analyzer

# **Technical Specifications**

# Technology

3-D Topographic Map Integral titanium incubation system Laser Scatter technology Flow Cytometry technology Cyanide Free reagents

#### Parameters

# Regular parameters:

WBC, RBC, HGB, HCT, MCV, MCH, MCHC, RDW-SD, RDW-CV, PLT, PDW, MPV, PCT, P-LCR, NEU#, NEU%, LYM#, LYM%, MON#, MON%, EOS#, EOS%, BAS#, BAS%

# Research parameters:

ALY#, ALY%, IG#, IG% 2 histograms, 2 scattergrams

# · Test speed:

60 samples/hour

### · Test mode:

CBC+5DIFF, CBC

# Performance

Item	Linear range	Carry over	CV
WBC	1.0-99.9×10 <sup>9</sup> /L	≤0.5%	≤2.0%
RBC	0. 3-7.0×10 <sup>12</sup> /L	≤0.5%	≤1.5%
HGB	20-240g/L	≤0.5%	≤1.5%
PLT	20-999×10 <sup>9</sup> /L	≤1.0%	≤4.0%



#### · QC and calibration

Multiple QC rules, including L-J, X-B etc.

Auto and manual calibration function for both whole blood mode and pre-diluted blood mode

# · Power Supply

100V-240V, 50Hz/60Hz


# Operation environment

Temperature: 15℃-30℃ Humidity: 30%-85%

Weight 53KG

Dimension 46cm(L)×57cm(W)×61cm(H)


# **Hemaray 83**5-part Hematology Analyzer

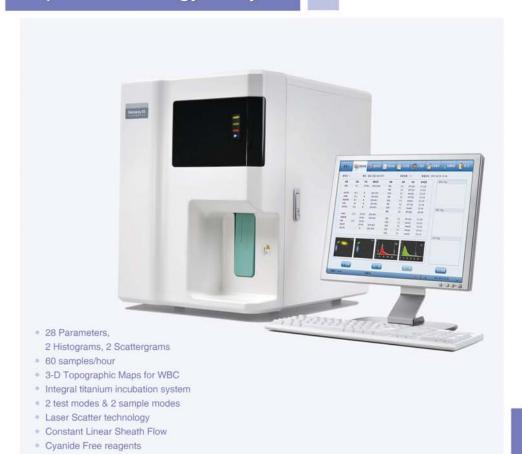




# **Hemaray 83**

# 5-part Hematology Analyzer

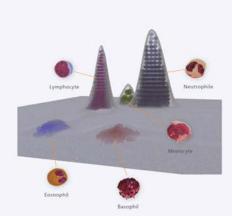



Cytomorphologically, is a useful way of 5-part differentiation of WBC according to the size, complexity and granularity of the cells. The multiple scatter light beams are used to detect the WBC, which named as the 3-DTopographic Map technology. The forward small angle laser beam, the forward large angle laser beam and the lateral laser beam stand for the different properties of the WBC.

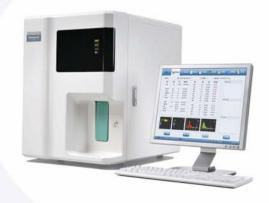

# **Constant Linear Sheath Flow**

A most important technology is to ensure the WBC passing the laser beam one by one, when the sheath flow passes the unique designed square counting chamber. As the diameter of the WBC will not be too different from the diameter of the transversal surface of the chamber, the WBC are fixed in the middle of the sheath flow passing the square chamber constantly with a high speed. This Constant Linear Sheath Flow ensures the accuracy of the WBC counting.

# Integral titanium incubation system

An integrated titanium incubation system is adopted to keep the reagent and the sample in the same constant temperature, which facilitates the complete reaction. This is the key technology to ensure the accuracy of 5-part differentiation.



# 2 test modes

CBC+5Diff mode CBC mode



Whole blood sample mode Pre-diluted blood sample mode





# **User-friendly software**

# Data management

- 100,000 results storage including histogram and scattergram
- · Multi print format including self-defined format
- . Mean, CV and SD values are calculated
- . Statistics including Sample, QC and Calibration
- Sort by sample number, patient name, type, time frame etc.

# Easy maintenance

- · Self-check during start up and shut down
- · Tubing automatic maintenance
- · Adjustable sleeping mode

# Multi-warning system

- · Reagent inventory check
- · Abnormal original data warning
- Multi warning flags

# System setup

- · Different levels of permission
- · Different reference ranges determined by age and gender

# LIS Interface

. LIS interface with HL7 protocol



